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EN NATURAL PEAT HOT WATER BOTTLE

HOT WATER BOTTLE filled with natural peat with 100% cotton cover. After warming up, it will provide comfortable heat for up to 3 hours, or up to 3 hours of coolness to relieve pain. Please follow the instructions for use in the manual. That way, the product will be a pleasant aid to you for a long time.

The product contains: natural peat 100%, PVC sheet (code 3), cotton slip (100% cotton, TEX, code 60)

The product cover contains: corrugated cardboard carton (PAP, code 20)

INSTRUCTIONS FOR USE:

Warm application: prior to warming up, remove the product from the carton and the cotton slip! After warming to the desired temperature (50 - 60° C), place the product back in its cotton slip. While warming up, we advise you to check the ideal temperature regularly by touch. Suitable heating methods are listed below. **DO NOT LEAVE THE PRODUCT UNATTENDED WHEN WARMING UP!**

Warming up in a microwave oven: fold the edges of the product down and put it on the rotating glass plate so that the product does not touch the oven walls. Warm it up for no more than 3 minutes at 600 Watts. If the product volume starts increasing, stop the warming immediately and check the cover for any damage. In that case, do not warm the product any more! The product may only be reheated after it has cooled down completely. If you require more frequent warming, warm it up in 10 second intervals until you achieve the ideal temperature (but no longer than 2 minutes altogether).

Warming up in a gas, hot-air or electrical oven: fold the edges of the product down and put it on a porcelain plate so that the product does not touch the oven walls. Warm it up for no more than 10 minutes at 150° C. If the product volume starts increasing, stop the warming immediately and check the cover for any damage. In that case, do not warm the product any more!

Warming up in a water bath: place the product inside a container filled with cold water. Warm it up to approx. 80° C. After achieving the temperature, turn off the heat and leave the pad to warm through in the water bath for another 5 - 10 minutes. Then remove the product and dry it. The water must not boil! That might cause irreversible damage to the product!

Cool application: remove the product from the carton and the cotton slip and place it in a refrigerator. Cool it down for about 2 hours. The cooling requires no supervision. **Do not place the product in freezers!**

STORAGE AND HANDLING: Before unpacking store at 5 - 25° C. Expanded product store in a cool and dry place without temperature oscillation (ideally 8 - 15° C). For longer-term storage without using the product, it is advisable to place it in a refrigerator. This will attenuate the drying of the product content (a natural process in the PVC cover). The product can be disposed of along with normal municipal waste.

IMPORTANT NOTICE:

The product may be damaged when in contact with sharp, pointed or hot objects or when overheated; the warranty does not cover such damage. The product was subjected to laboratory testing. No adverse effects were registered in contact with skin. If the peat blend gets in your eyes, rinse them thoroughly with cold water. If you are allergic or hypersensitive to the materials used, consult your physician prior to using the product.

APPROPRIATE USES:

Warm application: for example for pains and stiffness in muscles, joints and the spine, pain of non-inflammatory origin, period pain, subacute and chronic conditions of rheumatic origin, or to simply warm you up. **Caution: Do not use the product if you suffer from pain of inflammatory origin, circulatory disorders or fever conditions!**

Cool application: for example for bruises, sprains, headaches (migraine), toothache, insect stings, nosebleed, inflammations of muscles and joints, swellings and bruises.

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